



**Instruction for use** : Surface to be treated should first be compressed with dry gauze, remove the gauze carefully, and immediately cover the surface with Hemocoll. The amount required will depend on the force applied and severity of the bleeding. It is necessary to apply moderate pressure over the Hemocoll with a dry gauze.

Mfg. Lic. Number : 675

Manufactured & Marketed by:



**Advanced Biotech Products (P) Ltd.**

77, First Cross Street,  
Ragavan Colony, Chennai - 600083, India.

Under the Licence from :

**ENCOLL**  
Fremont, CA, USA

**Contact**

**For Andhra Pradesh**

Phone: +91 91769 99850

E-mail: andhra@abpdental.com

**For Rest of all India 7008.B**

Phone: 91 (44)-2489 1659

E-mail: info@abpdental.com

# HEMOCOLL™

**THE ABSORBABLE STERILE FIBRILLER COLLAGEN WOUND FILLER  
FOR  
DENTAL SURGERY  
HEMOCOLL IS NON TOXIC AND NON PYROGENIC**




**Advanced Biotech Products (P) Ltd**

www.abpdental.com

# HEMOCOLL™

Hemocoll is high purity type I reconstituted Collagen

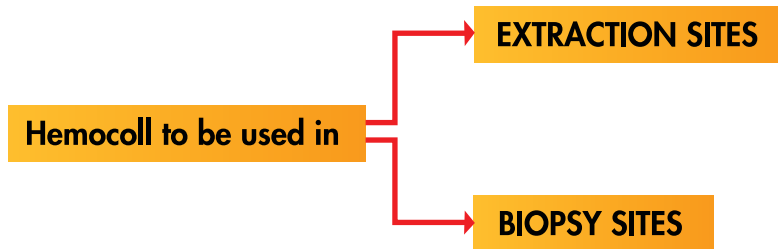

## Benefits of Hemocoll:

1. Controls bleeding and stabilizes blood clots (with in 2 to 5 mins).
2. Protects wound bed.
3. Helps in regeneration of lost Tissue.
4. Resorbable in 3-4 weeks.
5. Faster healing process.

## Available Sizes:

Individual Pack Sizes
0.25 gm (approx. 2.5 ml in Dry Fibrillar Form)
0.50 gm (approx. 5.0 ml in Dry Fibrillar Form)

## Precautions:

- ❖ Hemocoll is not to be used on the infected or contaminated wounds.
- ❖ Not intended for use in treating systemic coagulation disorders.
- ❖ Do not store above 32° C (90°F).
- ❖ Discard all open and un used dressing.
- ❖ Do not attempt to resterilize.
- ❖ Safety in pregnancy not established.